

DECENT SIMULATORS

CATHETERIZATION

TASK TRAINER | C0004

WHAT'S INCLUDED	CARRY BAG, USB, FEMALE GENITALS, MALE GENITALS (S), MALE GENITALS (L), MALE GENITALS WITH FORESKIN, ABSORBENT CLOTH, SILICONE PAD, LUBRICANT	
COMPANY DETAILS	UNIT 13D RATHENRAW INDUSTRIAL ESTATE ANTRIM, BT412SJ, UNITED KINGDOM W: WWW.DECENTSIMULATORS.COM	

PREPARATION

1. FILL THE BASE:

FILL THE BASE WITH WATER OR SIMULATED URINE UP TO THE OPENING. YOU CAN ADD AN ANTI-MOULD AGENT (LIKE MILTON STERILIZING FLUID) TO PREVENT MOULD GROWTH—JUST ENSURE IT'S SAFE FOR SKIN CONTACT.

2. SWAP THE INSERTS:

REMOVE THE EXISTING INSERT FROM THE BASE AND SLIDE IN THE NEW ONE, CAREFULLY TUCKING IN THE SIDE SILICONE. MAKE SURE THE MIDDLE HOLE LINES UP WITH THE RED CONNECTOR IN THE CENTER OF THE BASE.

3. LUBRICATE THE MALE INSERT WITH FORESKIN:

USE THE PROVIDED SYRINGE WITH SILICONE OIL AND THE GREEN INJECTOR. GENTLY INSERT IT UNDER THE SILICONE AND GRADUALLY APPLY SMALL AMOUNTS OF OIL. AFTERWARD, MOVE THE SILICONE SKIN A FEW TIMES TO EVENLY DISTRIBUTE THE OIL.

USAGE

1. LUBRICATE THE CATHETER:

USE OIL-BASED LUBRICANT AND ENSURE IT'S WELL-COATED BEFORE STARTING THE PROCEDURE.

2. EXTRA PREPARATION:

CONSIDER INJECTING A SMALL AMOUNT OF LUBRICANT INTO THE SILICONE INSERT'S URETHRA BEFORE TRAINING.

STORAGE

1. CLEANING:

WASH ALL PARTS IN THE SINK USING SOAP AND WATER.

2. DRYING:

MAKE SURE EVERY PART IS COMPLETELY DRY BEFORE STORING TO PREVENT MOLD GROWTH.

REPAIR INSTRUCTIONS

- PLASTIC PARTS: REPAIR USING ANY CYANOACRYLATE GLUE (E.G., 'SUPERGLUE').
- SILICONE PARTS: REPAIR USING SILICONE GLUE (E.G., 'SIL-POXY'). FOR ADDED STRENGTH, APPLY NYLON MESH OVER THE DAMAGED AREA.

REPAIR KITS ARE AVAILABLE AT WWW.DECENTSIMULATORS.COM/STORE OR THROUGH YOUR LOCAL DISTRIBUTOR.